

CRC®-55tW Dose Calibrator & Well Counter

For Speed and Accuracy in Measuring Isotopes



Features

- · 8 in. VGA touch screen color display
- USB/PC Communications
- · Software upgrade via USB or flash drive
- USB printer capability
- · Chamber plug-and-play capability
- 28 programmable nuclide keys
- Built-in dose calibration, quality control and self-diagnostics
- Over 80 nuclides in memory
- Automatic chamber QC including linearity
- Hard copy records of data available from optional printer
- Compatible with nuclear medicine information management systems via USB
- 256 channel MCA with detailed spectrum for identification analysis
- · Well counter for wipe tests
- User-definable protocols for wipe testing
- Automated well QC including Chi-Square and MDA
- · Manual and automatic ROI
- Includes a pre-set key for F-18 measurements
- Reports: Well counter reports for autocalibration, system test, MDA, and Chi-Square are archived and searchable by date, type, and activity
- Automated linearity and geometry testing

Description

As a combination of the CRC®-55tR Dose Calibrator and Well Counter, the CRC-55tW Dose Calibrators provide advanced features with the speed and accuracy you need to measure activity and prepare doses. Its ion chamber is one of the time-tested, high pressure well designs by Capintec™ capable of measuring a dose as high as 6 Ci Tc99m (250 GBq) with high accuracy. The CRC-55tW calibrator features a new helpful chamber plug-and-play capability.

For wipe testing, the CRC-55tW calibrator allows the user to define specific counting procedures (protocols) with trigger levels for work, patient, unrestricted areas and sealed source leak tests. The CRC-55tW unit also performs counting functions for wipe tests in as little as six seconds at activities as low as 1 nCi. Low activities are measured with a drilled well high sensitivity Sodium lodide (NaI) detector.

The well counter includes a 256 channel MCA which provides detailed spectrum for identification and analysis. Manual and automatic ROI selection are available.

The chamber and counter of the CRC-55tW calibrator are combined in a menu-driven, touch screen interface that is easy to learn and use.

Reports software, new for the CRC-55t-W unit, archives well counter data for quality assurance procedures, and wipe measurements. Well counter reports are stored and searchable by date range for later viewing or printing. Wipe tests are searchable by date range, wipe type, and activity.

Count on us for excellence in energy measurement and for our unsurpassed customer service, training and support!

Capintec | CRC-55tW Dose Calibrator

Optional Componets



Specifications

Physical:

· Console Dimensions:

Height: 24 cm (9.5 in.)
Width: 23 cm (9.0 in.)
Length: 27 cm (10.5 in.)
Weight: 3.4 kg (7.5 lb)
Chamber Dimensions:

Height: 43.8 cm (17.25 in.)

Diameter: 17.2 cm (6.76 in.)

Weight: 13.6 kg (30 lb)

Well Diameter: 6.1 cm (2.4 in.)
Well Depth: 25.4 cm (10.0 in.)
Cable Length¹: 3.7 m (12 ft)
Well Counter Dimensions:

Height: 23.8 cm (9.38 in.)

Diameter: 15.2 cm (6 in.)

Weight: 6.9 kg (15.2 lb)

Well Diameter: 1.7 cm (.67 in.)

Well Depth: 3.8 cm (1.5 in.)
Cable Length: 2.7 m (9 ft)

· Well Detector:

Type: Sodium Iodide (NaI) drilled-well crystal detector 256 Channel MCA, manual and automatic ROI

Warning Trigger Levels: User-definable

Tests: Daily test, energy calibration, and reproducibility

Enhanced QC includes Chi-Square and MDA

1: Longer cables are available. Consult factory.

· Ionization Chamber:

Fill Gas: 12 atm Ultra-Pure Argon **Measurement:** Auto ranging

Activity Range:

6 Ci (222 GBq) Tc99m max 2 Ci (74 GBq) F18 max

Resolution: .01 µCi (.001 MBq), max.

Response Time: Within 2 seconds, for low activity

sample, 4 to 16 seconds

Tests: Daily tests include Auto Zero, Background Adjustment, Voltage Test, Data Check, Accuracy and Constancy Enhanced QC includes linearity, geometry, strip QC.

· Display Screen:

Type: 8 in. VGA LCD color touch screen display

Bq/Ci Reading: User selectable or fixed

Activity Display: Selected radionuclide, calibration number, measured activity and display units (Bq/Ci)

Count Rate Values: Wipe results

· Electrometer:

Accuracy: Better than ± 2% **Linearity:** Within ± 2%

· Repeatability of Measurement:

Within \pm 1% within 24 hours during which time the calibrator is powered at all times

Nuclear Data:

Nuclide Setting Keys: 28 programmable keys

System Memory: Over 80 nuclides

(cal number and half-life)

A pre-set key measures up to 2.0 Ci (74.0 GBq) of F-18



Capintec | CRC-55tW Dose Calibrator

Optional Componets



· Standard Source Data:

System Memory: Co-57, Co-60, Ba-133, Cs-137

Molybdenum-99 Assay:

Methods: CapMac and canisters

Measured Values: Mo-99 elution, Tc-99m,

Tc-99m/Mo-99 ratio

· PC Port:

Interface: RS-232 and USB

Compatibility: Standard Nuclear Medicine

Management Systems

Printer:

Interface: RS-232 and USB

Type: Epson Roll, Epson Slip or Okidata dot matrix

· Power Requirements:

100-240 V ac (50/60 Hz) 100 MA

Tests:

Diagnostics: Full test of program, system memories

· Cable:

Printer: (Optional): 1.8 m (6 ft)

Power: 1.8 m (6 ft)

Ordering Information

5130-2216 CRC-55tW Dose Calibrator

5420-2072 Auxiliary Shield 7300-2004 Chamber Well Insert 5430-0058 Epson Roll Printer

7120-1199 CRP-200 Dose Tickets & Labels

5430-0100 Epson Ticket Printer 7300-2450 Environmental Shield

7300-2005 Dipper





capintec.com